

# Mounting / Demounting of Temples

## General


### Click Off

1. Check opening direction of hinge
2. Hold frame at a vertical angle
3. Secure hinge firmly with finger
4. a) Remove temple with a jolt off the hinge or  
b) lever temple off the hinge  
(again, mind the opening direction of the hinge with both methods)

### Click In

1. Check opening direction of hinge
2. Secure hinge firmly with finger
3. Click in the temple with light pressure

## Methods

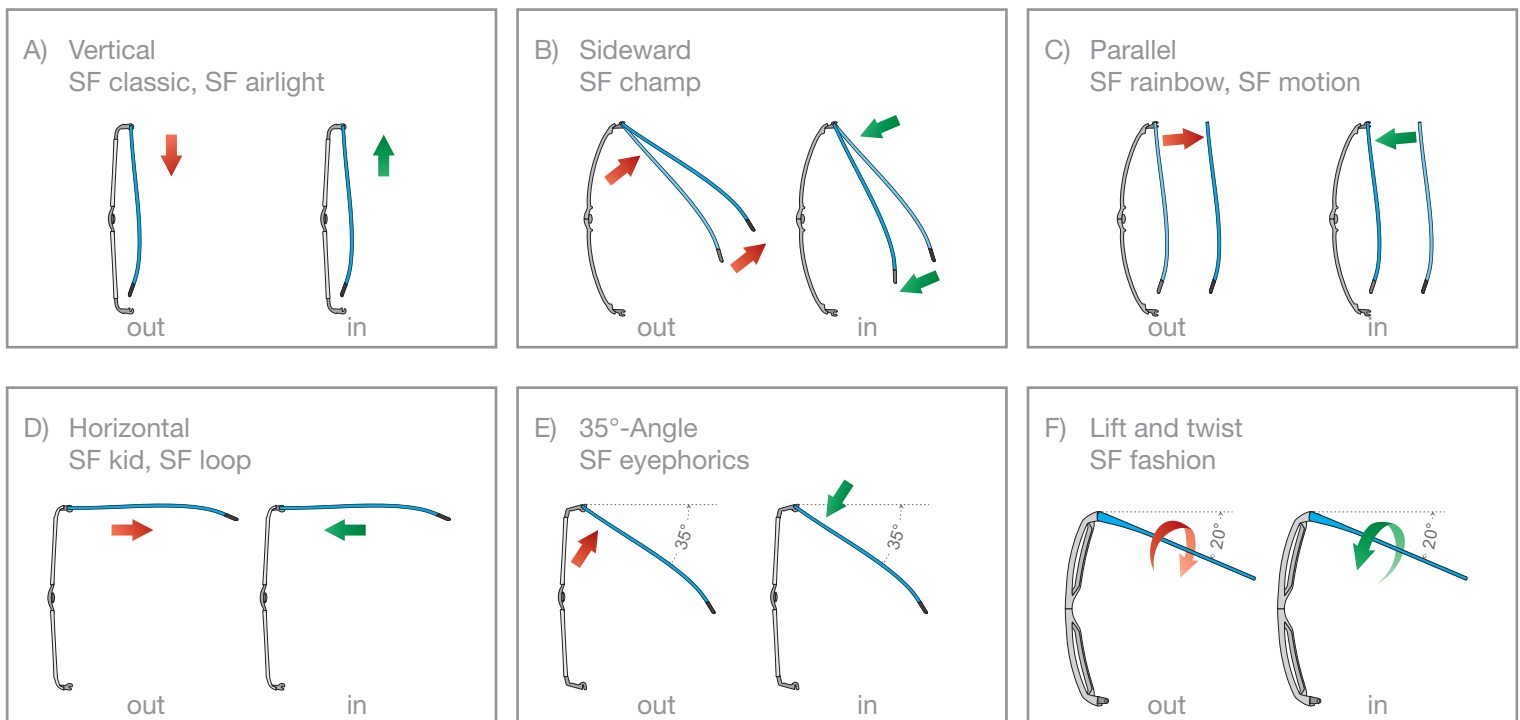

## Particularities and Tips

- Mounting and demounting of temples is strong on purpose and serves the tight hold
- Protection of overexpansion for models SF loop, SF champ, SF kid
- Removing temple with a jolt is important, otherwise the material can be overexpanded
- Older hinges may vary – please check opening direction of hinge
- Demonstration videos are provided in the login-area of our homepage

### Important:

**Mounting and demounting of temples are to be executed by trained opticians only not by end-consumers.**



like  
wearing  
nothing.

**+**  
perfect fit  
modular individuality  
100% swiss-made

# Mounting / Demounting of Nosepads (for Opticians)

## General

### Click Off

1. Position frame with nose pad showing downwards
2. Press wings down with both thumbs until pad clicks off

### Click In Options

#### A) Without Aid

1. Insert pad notch from the bottom with fingernails, press over bolt on the bridge until it clicks in

#### B) With mounting tool pad clip (blue)

1. Position pad onto flat surface, wings showing upwards
2. Press tool vertical onto pad and lock it
3. Press tool vertical onto bolt on the bridge and click it in with light pressure

#### C) With mounting tool pad clip (black) – works without temples only

1. Place pad onto tool, wings showing down
2. Insert black V-notch into hole of tool for fixation
3. Press front with thumb placed onto center of bridge vertical down until pad clicks in

## Important

- Nose pads will not be damaged, provided mounting & demounting are handled correctly
- The strong click in and click off procedure is on purpose and serves a better hold of nose pad



like  
wearing  
nothing.

**+**  
perfect fit  
modular individuality  
100% swiss-made